# EmbryoGen®/ BlastGen™

Product No.:

1206

Consisting of:

1204 and 1205

## Languages:

bg	cs	da	de	el
en	es	et	fr	hr
hu	is	it	kk	lt
lv	mk	nl	no	pt
ro	ru	sk	sl	sv
tr	uk			

## Customer Service:

E-mail: customer.service@origio.com  
Tel.: +45 46 79 02 02 Fax: +45 46 79 03 02

ORIGIO a/s  
Knardrupvej 2, DK-2760 Måløv, Denmark  
www.origio.com  
Tel: +45 46 79 02 00 · Fax: +45 46 79 03 00

# origio

a CooperSurgical Company


## Ingredientia

### 1204 EmbryoGen®

Acida aminica: Ala, alanylglutamin, Arg, Asn, Asp, Cys, Glu, Gly, His, Ile, Leu, Lys, Met, Phe, Pro, Ser, Thr, Trp, Tyr, Val, Taurinum; Acidum edeticum; Albumini humani solutio; Aquae; Gentamicinum; Glucosum; Hyaluronas; KCl; Lactas; MgSO<sub>4</sub>; NaCl; NaH<sub>2</sub>PO<sub>4</sub>; Na<sub>2</sub>HPO<sub>4</sub>; NaHCO<sub>3</sub>; KH<sub>2</sub>PO<sub>4</sub>; Natrii citras; Pyruvas; Sargramostimum; Vitamini: B<sub>1</sub>, B<sub>2</sub>,

### 1205 BlastGen™

Acida aminica: Ala, alanylglutamin, Arg, Asp, Asn, Cys, Glu, Gly, His, Ile, Leu, Lys, Met, Phe, Pro, Ser, Thr, Trp, Tyr, Val; Albumini humani solutio; Aquae; Gentamicinum; Glucosum; Hyaluronas; K<sub>2</sub>SO<sub>4</sub>; Lactas; MgSO<sub>4</sub>; NaCl; NaH<sub>2</sub>PO<sub>4</sub>; KCl; KH<sub>2</sub>PO<sub>4</sub>; Na<sub>2</sub>HPO<sub>4</sub>; Pyruvas; SSR; Sargramostimum; Vitamini: B<sub>1</sub>, B<sub>2</sub>, B<sub>5</sub>, B<sub>6</sub>, B<sub>9</sub>, Cholinum, Inositolum, Nicotinamidum, Pyridoxinum

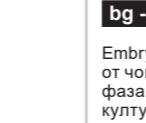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#### en - english

EmbryoGen® is for the culture of human embryos until the 2-8 cell stage. BlastGen™ is for the culture of 4-8 cell stage human embryos to the blastocyst stage. BlastGen™ may also be used for embryo transfer.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

#### Package and pack size

EmbryoGen® (1204): 1 x 3 ml  
BlastGen™ (1205): 1 x 3 ml

#### Contains

Human albumin solution (HAS)  
Gentamicin sulphate 10 µg/ml  
Sargamostim (GM-CSF) 2 ng/ml  
Recombinant human insulin (only BlastGen™)

#### Quality control testing

Sterility tested (Ph.Eur., USP)  
Osmolality tested (Ph.Eur., USP)  
pH tested (Ph.Eur., USP)  
Endotoxin tested < 0,15 EU/ml (Ph.Eur., USP)  
HSA analysis (Ph.Eur., USP)  
Mouse Embryo Assay (MEA) tested

**Note:** The results of each batch are stated on a Certificate of Analysis, which is available at [www.origio.com](http://www.origio.com).

#### Storage instructions and stability

The products are aseptically processed and supplied sterile.  
Store in original container at 2-8°C, protected from light.  
Do not freeze.

Discard excess (unused) media following warming.  
The product is to be used within 7 days after opening.

When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

#### Precautions and warnings

Do not use the product if:  
1. Product packaging appears damaged or if the seal is broken.  
2. Expiry date has been exceeded.  
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

**Caution:** All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

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