Liquid Paraffin

Product No.:

1010

1010

Languages:

bg	cs	da	de	el
en	es	et	fr	hr
hu	is	it	kk	lt
lv	mk	nl	no	pt
ro	ru	sk	sl	sv
tr	uk			

Customer Service:

E-mail: customer.service@origio.com
Tel.: +45 46 79 02 02 Fax: +45 46 79 03 02



ORIGIO a/s
Knudrupvej 2, DK-2760 Måløv, Denmark
www.origio.com
Tel.: +45 46 79 02 00 Fax: +45 46 79 03 00

origio

a CooperSurgical Company

Ingredientia
1010 Liquid Paraffin
Paraffinum liquidum

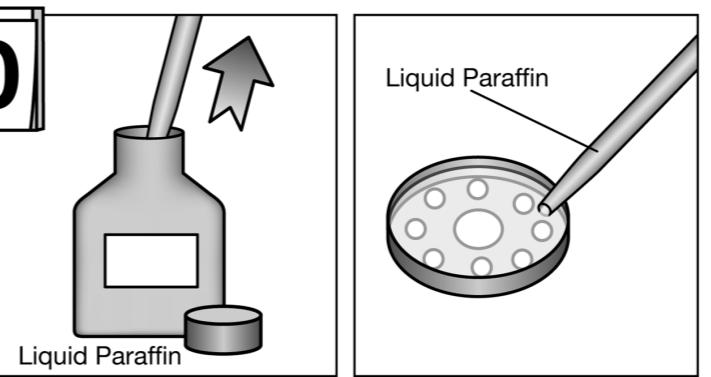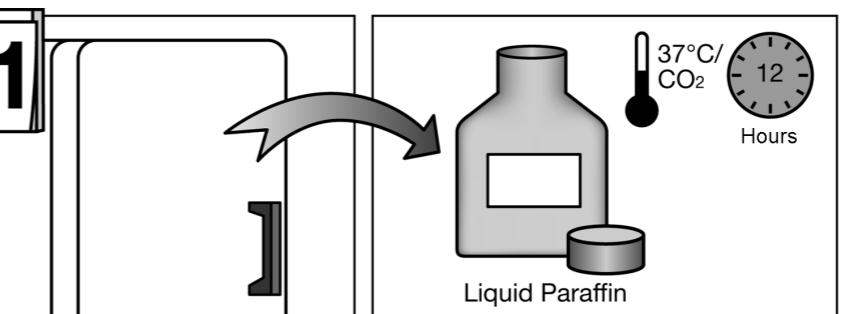1010 Liquid Paraffin
Paraffinum liquidum

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en - english

Liquid Paraffin is used as oil overlay for culture media during IVF and ICSI procedures.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Quality control testing

Sterility tested (Ph.Eur., USP)

Endotoxin tested ≤ 0,1 EU/ml (Ph.Eur., USP)

Mouse Embryo Assay (MEA) tested

Note: The results of each batch are stated on a Certificate of Analysis, which is available at [www.origio.com](#).

Storage instructions and stability

The products are aseptically processed and supplied sterile.

Store in original container at 2-8°C, protected from light.

Do not freeze.

Discard excess (unused) media following warming.

The product is to be used within 28 days after opening.

When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings

Do not use the product if:

- Product packaging appears damaged or if the seal is broken.
- Expiry date has been exceeded.
- The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

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During the manufacturing process, the product is washed with medium containing small amounts of human serum albumin.

Caution: All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

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Instructions for use

1. Place the Liquid Paraffin in an incubator with a 5% CO₂ environment at 37°C for at least 12 hours prior to use.

2. Pipette the intended volume of medium to be used for the microdrop on the bottom of a Petri dish.

3. Gently pipette the equilibrated Liquid Paraffin into the Petri dish until the microdrops are sufficiently covered with oil.

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